
		Instructions For Use			
Document No.	BMB/IFU/32	Rev No. & Date	00 & 15/11/2017	Issue No & Date	01 & 15/11/2017

Zebra/Stripped Guide wire/Hydrophilic Guide wire/PTFE Guide wire/Stainless Steel Wire

Device Description

Guide wire is a thin, usually flexible wire that can be inserted into a confined or tortuous space to act as a guide for subsequent insertion of a catheter/stent/dilators.

Intended Use

Guidewire is a device used to direct a catheter/stent/dilators to the desired anatomical location during diagnostic or interventional procedures.

Contradictions

None Known

Warnings

1. Risk of Reuse: Reuse of the product causes infections. The guide wire might kink with reuse.

Precautions

1. The Guidewire should be used by a physician, who is well trained in manipulation and observation of guide wires under fluoroscope.
2. Do not resterilise & use only if packaging is intact.
3. Guidewire is normally intended for continuous use for more than 60 minutes and less than 30 days. Life time of the device is 1 day
4. The packaged product must be stored at a temperature of between 23⁰C & 33⁰C. Protect the packed products from direct sunlight. With Proper Storage the packed product may be used until the expiry date given on the packaging.
5. To be opened in sterile room only. For Single use only.
6. The product & package should be disposed of safely & properly after use, as per bio-medical waste management
7. Product is compatible with other standard medical devices
8. Standard interventional procedure should be employed while inserting the catheter






BIORAD MEDISYS PVT. LTD.

"Biorad House", No. 660, Eshwari Industrial Estate, Hulimavu
Bannerghata Road, Bangalore-560076, India.

Customer Care: +91-80-65658007, Telefax: +91-80-26480683/4

E-mail: biorad@bioradmedisys.com, Website: www.biorad-medisys.com

 		Instructions For Use			
Document No.	BMB/IFU/32	Rev No. & Date	00 & 15/11/2017	Issue No & Date	01 & 15/11/2017



CMC Medical Device & Drugs S.L

C/Horaico Lengo 18, CP 29006, Malaga Spain

Tel: +34951214054, Fax +34952330100








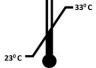





Movil: +34661416622

Info@cmcmedicaldevices.es

www.cmcmedicaldevices.com

Labeling Symbols

The following is an explanation of symbols that may be used on your product packaging

	Consult instructions for use
	Keep Away from Sunlight
	Lot Number
	Date of Manufacture
	Use By
	Keep dry
	Do not Resterilize
	Temperature Limit
	Do not re-use
	Sterilized using Ethylene Oxide
	Do not use if Package is Damaged
	Manufacturer
	Authorized representative in the European Community